





# Now Indicated for Traumatic Injuries of the Descending Thoracic Aorta

## Low Profile without compromise

- ▶ Developed to enable a controlled and accurate stent-graft deployment in the proximal landing zone
- ▶ Designed to provide complete seal apposition

*New 22mm device now available for this indication*

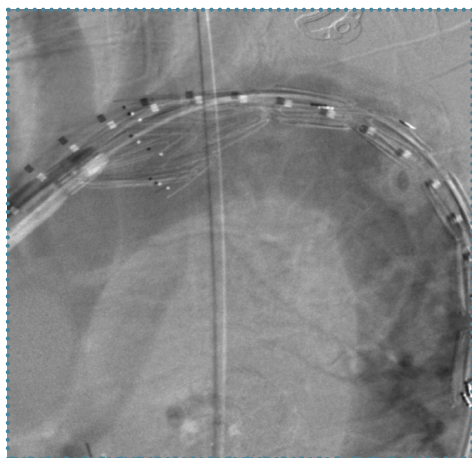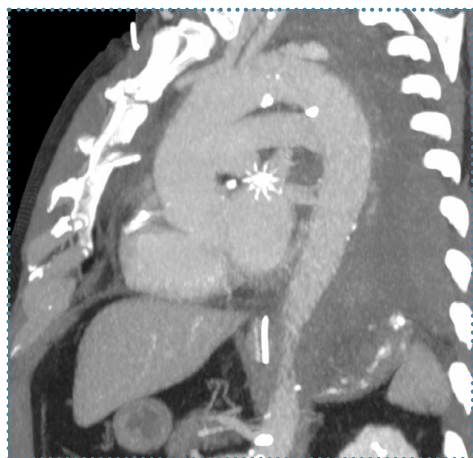

Images Courtesy of Dr. Benjamin Starnes, The University of Washington.

## Excellent acute performance in highly complex patient population

RelayPro was studied in a multicenter, unblinded, non-randomized, prospective IDE clinical study of 50 subjects to evaluate traumatic injuries of the descending thoracic aorta. The patient population averaged an Injury Severity Score of 30.3 ( $\pm 16.3$ ) and 88% grade 3/4 injuries.

## Key outcomes through 12 months

**2%**

All-Cause Mortality

**0%**

Stroke

**2%**

Paralysis

**98%**

Technical Success\*

**2%**

Type Ia Endoleaks

**0%**

Type Ib Endoleaks

**98%**

Patency

**2%**

Stent-graft thrombosis

**No**

Migrations

**No**

Fractures

**No**

Conversions

**No**

Ruptures


\*Technical Success at the time of the index procedure, defined as successful delivery and deployment of the device, including withdrawal of the delivery system.

*All patients should be advised that endovascular treatment requires yearly lifelong imaging and regular follow-up to assess their health and the performance of their endovascular graft.*

Bare Stent		
Stent-Graft Diameter	Proximal Length	Distal Length
22 - 28mm	15mm	20mm
30 - 38mm	20mm	20mm
40 - 46mm	25mm	20mm

Non-Bare Stent		
Stent-Graft Diameter	Proximal Length	Distal Length
22 - 38mm	25mm	20mm
40 - 46mm	30mm	20mm





At Terumo Aortic, we are 100% focused on  
addressing every segment of the aorta,  
from the aortic root to the iliacs.

**terumoaortic.com**

Discover solutions for every segment of the aorta

Consult Instructions for Use, [eifu.terumoaortic.com](http://eifu.terumoaortic.com), for more information on use, indications, contraindications and warnings/precautions.

Product availability subject to local regulatory approval.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

 Manufactured by: Bolton Medical Inc, 799 International Parkway, Sunrise, Florida 33325, USA

USA\_0259-A Apr23 • PM-06835

**TERUMO**  
Aortic